



SOL-M™ Blunt Fill Needle

with and without filter

Usage Guidelines

Blunt Fill Needle - for medication vials.

Insert the needle into the centre of the vial stopper at a 90° angle. This will help reduce the risk of rubber fragments from the side of the vial stopper contaminating the medication.

The Blunt Fill Needle is for **single use only**. Reuse will dull the needle, remove the silicone coating and increase the chance of coring and rubber fragmentation.

Blunt Fill Needle with 5 micron filter - for medication ampoules.

Draw up medication from glass ampoule. The 5 micron filter will aid preventing glass particles to be aspirated to the syringe.



18 Gauge


45° Bevel




● **Red** – Blunt Fill Needle

● **Purple** – Blunt Fill Needle with Filter

Blunt Fill Needle

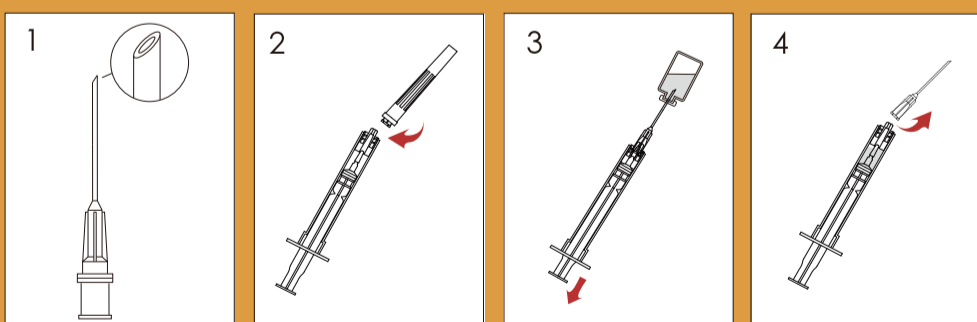

Blunt Fill Needle with Filter

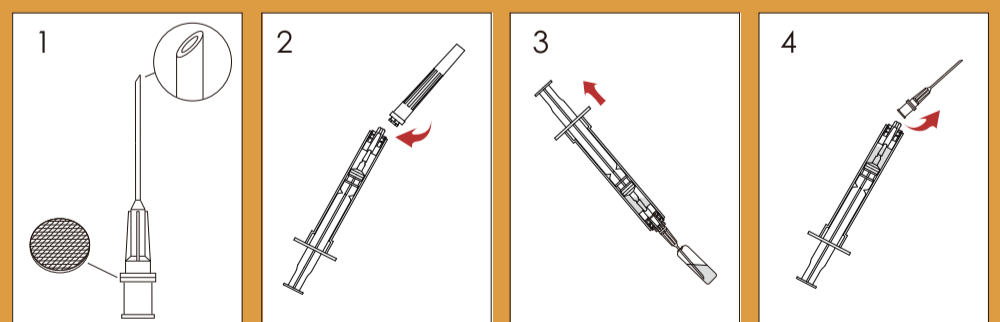

For more information, please contact:

Sol-Millennium Europe Sp. z o.o.
Twarda 18, PL-00105 Warsaw, Poland
Tel: +48 22 202 67 23, e-mail: info.EU@sol-m.com

sol-m.com

EMA-BF-009-0522



SOL-MILLENNIUM®
Building a Healthier Tomorrow