

BD MaxPlus™ pressure rated extension set with needleless connector

INDICATIONS FOR USE: The pressure rated extension set with needleless connector allows the user to add medication into the primary line without the use of a needle. The pressure rated extension set with needleless connector may also be used with low-pressure power injectors rated up to 325 psi.

DIRECTIONS: Use Aseptic Technique.

1. Peel open package and remove set. Verify all connections.
2. Prime extension set per facility protocol taking care to invert connector to expel air.
3. Remove protective cap and attach primed extension set to desired venous access device.
4. Prior to every access, swab connector per facility protocol.
5. Administer by attaching fully primed syringe or administration set to connector. Insert male luer into connector turning clockwise until tight.
6. After use, flush set per facility protocol.
7. For subsequent connections, repeat from step 4.


WARNINGS AND CAUTIONS:

- Pressure rated up to 325 psi at a flow rate of 10 mL/second.
- For use with pressure, this product must be secured to other luer lock devices also rated for use with pressure up to 325 psi.
- Do not use needles, blunt cannula, or non ISO standard luers to access the connector.
- Trace all lines prior to administration.
- Disconnect IV Set or syringe from connector prior to clamping the catheter or line.
- The device should be changed according to facility protocol or in accordance with current recognized guidelines for IV therapy.
- Federal (USA) law restricts this device to sale by or on order of a physician or other licensed practitioner.



Rx Only

DEHP or Natural Rubber Latex are not part of the material formulation.

 BD Switzerland Sarl, Route de Crassier 17, Business Park Terre-Bonne, Batiment A4, 1262 Eysins Switzerland
Distributed by CareFusion San Diego, CA USA 1.800.854.7128
Made In Mexico
630-02130 Rev 01
MaxPlus is trademark of Becton, Dickinson and Company.

Symbols Glossary

<http://www.bd.com/symbols-glossary>



Do not re-use



Do not use if package is damaged



Do not re-sterilize



Non-pyrogenic



Sterilized using irradiation



Consult instructions for use



Caution



Manufacturer



Keep away from sunlight



Fragile, handle with care



Temperature limit



Keep dry



Catalogue number



Batch code



Use-by date