

MISI

Medical Imaging Solutions International

CT ANGIOGRAPHIC SYRINGE

INSTRUCTIONS FOR USE

Please read the product instructions carefully to ensure safe and proper operation of this product.

Important safety precautions: This appliance is intended for use by personnel with experience in diagnostic angiographic procedures who have received appropriate training.

Compatible injector models:

This product is designed for use in contrast media injection procedures. It is compatible **only** with the contrast media injection systems from **Medrad® MCT®, Medrad® MCT Plus®, Medrad® Vistron CT®** and **Medrad Envision CT®**. This appliance is **intended for single use only**.

CONTRAINDICATIONS

This product is not intended for multiple purposes. It must not be used for drug injection, chemical treatment, or any other application not explicitly specified in the product instructions.

Compatibility: Equipment models Compatible with 100200A syringe and with **Medrad® Spectris®** and **Medrad® MCT®, Medrad® MCT Plus®, Medrad® Vistron CT®** and **Medrad Envision CT®** injectors.

REF	Product Description	Syringe Model #	Syringe Specifications
100101	CT Angiographic Syringe	100200A	200mL Syringe

INTENDED USE

The **CT Angiographic Syringe** is intended for the injection of contrast media or physiological saline. This syringe is for **single use only** and is designed to be used with angiographic injectors that are legally marketed in the United States.

The syringe seal and connecting tubing are designed to withstand up to 350 psi of pressure without leaking when used with high-pressure injection equipment

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a licensed physician.

WARNINGS

- Do **not** use a standard (common) intravenous needle. It is strongly recommended to use a **19G to 22G** high-pressure-resistant intravenous needle or a high-pressure-resistant central venous catheter. Failure to do so may result in bursting or leakage during injection.
- Risk of Air Embolism:** Air bubbles (air emboli) can cause serious injury or death. To minimize the risk, all air must be fully expelled from the syringe and connected tubing before injection.
- Single Use Only: Do not reuse** disposable products. Reuse or failure to follow proper aseptic technique may lead to **biological contamination** and pose serious risks to patient safety. **Discard the product immediately after use.**
- Risk of Contamination: Do not remove the piston** from the syringe. Doing so may lead to bacterial contamination of the injector, increasing the risk of patient infection. Always fill the syringe **according to the instructions for use**, without removing the piston.
- Do Not Store Filled Syringes:** Using the syringe to store contrast media may result in **bacterial contamination**. After filling, the syringe must be **used immediately**. Do **not** store a pre-filled syringe for later use. Any filled but unused syringe must be **discarded**.
- Do Not Use Damaged Packaging or Components:** If the packaging is **opened, damaged**, or if any component is **broken**, use of the product may result in injury to the **patient or operator**. Always **visually inspect** the packaging and contents **prior to each use** to ensure integrity.

MISI

Medical Imaging Solutions International

- If a syringe is not well-sealed, it may lead patient's injury. Unless the syringe is well sealed, otherwise please do not fill or take injection. Please make sure the alignment marks on the injector and injection head are matched-up, the piston and the plunger has been inter-locked. Bad sealing will cause thrombosis bubbles or injection dose reducing.
- **Ensure Proper Sealing:** If the syringe is not properly sealed, it may cause injury to the patient. Do **not** fill or inject unless the syringe is fully sealed. Before use, ensure that the **alignment marks on the injector and injection head are properly matched** and that the **piston and plunger are securely interlocked**. Improper sealing can lead to air bubbles (thrombosis risk) or reduced injection dosage.
- **Risk of Leakage and Tube Damage:** Contrast media leakage or connecting tube breakage may cause injury to the patient or operator. Ensure the tubing is **clear and unobstructed** before use. Do **not exceed** the pressure limit specified on the package. Excessive pressure or a blocked tubing line can result in connector failure, leakage, or tube rupture.
- **Sharp Needle Hazard:** Using sharp needles may cause injury to the operator. Handle and insert the needle into the bottle with care to avoid accidental injury

CAUTIONS

- Familiarize yourself with and fully understand the product specifications before use.
- Do not use products beyond their designated expiration dating.
- After opening the package, inspect for content accuracy, and verify there is no damage or missing parts.
- Check the connection tubing to ensure it is free from twists, kinks, or blockages before injection.
- Improper installation of components may cause damage. Ensure all fittings are secure but not over-tightened to minimize the risk of leakage, breakage, or component failure.

INSTALLATION AND APPLICATION

Installation Instructions

- Ensure the pushing head of the high-pressure injection system is fully retracted.
- Align the flange of the syringe with the groove on the injection equipment head.
- Rotate the syringe **one-quarter turn clockwise** until it is securely locked in place.

Contrast Media Filling Method:

1. Manually turn the spin handle of the high-pressure injection equipment to expel all air from the syringe.
2. Remove the protective cap from the syringe cone tip. Under aseptic conditions, securely connect the J-shape tube to the syringe cone tip.
3. Insert the J-shape tube aspirator into the bottom of the contrast media bottle. Use the injection equipment controls to draw the required amount of contrast media into the syringe.
4. Remove the J-shape aspirator. Connect one end of the spiral tube to the syringe cone tip and turn lightly to secure. Connect the other end of the spiral tube to the scalp needle or IV tubing.
5. Turn the tail knob of the high-pressure injection equipment to slowly push the contrast media toward the connection point. If the contrast media does not reach the connection, a bubble may be present in the tubing. In this case, manually move the piston from the 3 mL position back to the 5 mL position, then gently shake the injector head to allow bubbles to rise and be expelled. **Do not tap or knock the syringe to remove bubbles.**

MISI

Medical Imaging Solutions International

TROUBLESHOOTING

Leakage at the Fitting During High-Pressure Injection:

6:100 luer seal fittings are used in the connecting tubing between the syringe and scalp needle (or IV). These fittings **must be securely tightened** during high-pressure injection to prevent minor leakage.

Leakage at high pressure syringe tail:









Repeated use of the syringe can lead to crystallization of the contrast media and partial blockage of the liquid flow path. This blockage can cause a sudden increase in pressure, which may result in leakage at the syringe tail. **To prevent this issue, do not reuse syringes**

STORAGE CONDITIONS

Store this product in a clean, dry area with good air circulation. Make sure the humidity stays below 80%. Do not store it in places where there are corrosive gases or chemicals in the air, like strong cleaners or industrial fumes.

MISI

Medical Imaging Solutions International

Symbol	Standard Reference	Standard Title	Symbol Title	Explanation
	21 CFR Part 801.109(b)(1)	Labeling-Prescription devices	Prescription only	Indicates that the product is authorized for sale by or on the order of a licensed healthcare practitioner.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Batch Code	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Do not re-use	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Sterilized using ethylene oxide	Indicates a medical device that has been sterilized using ethylene oxide.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Catalogue Number	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Use-by date	Indicates the date after which the medical device is not to be used.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, but presented on the medical device itself.
	ISO 15223-1	Medical Devices – Symbols to be used with Medical device labels, labelling and information to be supplied.	Consult instructions for use	Indicates the need for the user to consult the instructions for use.

Manufactured for:

CHS USA Incorporated
 200 Kelly Dr., Suite A
 Peachtree City, GA 30269 USA
<http://www.chsusainc.com>

Made in Canada

REVISION HISTORY

Rev. #	DR #	Date	Description
01	118676	08/18/2025	• New IN-100200A R01 CT Angiographic Syringe IFU as outlined in CO #18226 / CO Task #118664.